



**EZ-ON PRODUCTS**

**1-800-323-6598**

**EZ-ON**

**www.ezonpro.com**

# 43YC

CAR CAM WITH CROTCH STRAP

## IMPORTANT INFO

- Meets FMVSS213
- 43YC is for use by a passenger weighing 31 – 106
- You must use EZ-ON hardware tether kit (included) to install this conversion vest if the factory installed tether anchor weight limit is less than the weight of the passenger or if there is no tether anchor point available in your vehicle.
- Register this product to be notified for a safety recall

## WARRANTY

*EZ-ON Products stands behind the products we make. Warranty covers any defects in material or workmanships under normal use during the warranty period. We will repair or replace product defects for 1 year from manufacturer date.*

## WARNING

Death or serious injury can occur if manufacturer's instructions are not followed correctly. All EZ-ON products are dynamically tested to meet NHTSA standards. Please follow washing instructions on product tag. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement after six years. EZ-ON Products are not liable for injury if these recommendations are not followed.

## EMERGENCY RELEASE PROCEDURE

When evacuating a passenger in an 43YC belt, one cut of the lap belt will allow the passenger to evacuate the vehicle.

### CUT ALONG DOTTED LINE FOR EMERGENCY RELEASE



**100 SCBC  
EZ-ON Safe Cut  
Belt Cutter**










# 43YC

## CAR CAM WITH CROTCH STRAP

### INSTALLATION INSTRUCTIONS

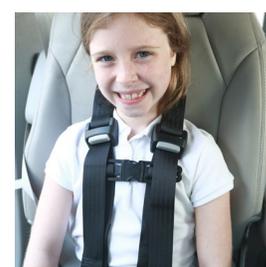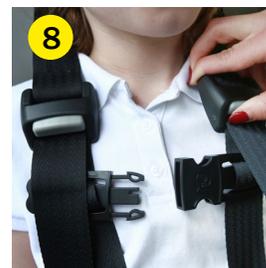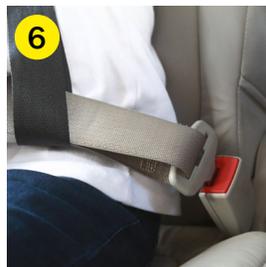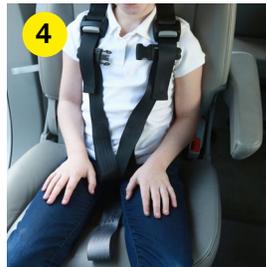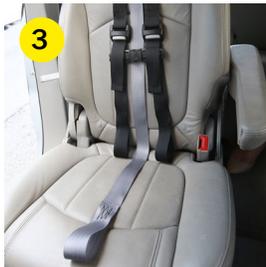

1. Locate the tether anchor bracket.

\*Check the vehicle owner's manual to locate factory installed tether bracket.

\*43YC tether strap can be turned one half twist to snap on difficult anchor locations.

2. Connect tether hook to tether anchor bracket directly behind passenger. Twenty degrees from "Y" center is acceptable.

\* Tighten by pulling on black webbing so Y-bridge sits on seat top.

3. Prior to the child sitting down, place all of the straps on the front of the seat. Make sure the gray strap is laying across the bottom of seat cushion.

4. Sit passenger on the gray strap, so that it extends forward from between the passenger's legs. Place the adjustable black straps in front of the passenger's body.

5. Thread the vehicle lap belt through the first chest strap, then the gray crotch strap, and then through the second chest strap.

6. Buckle lap belt and tighten.

7. First adjust the gray crotch strap from the top of the seat, by pulling on the loose webbing.

8. Adjust the shoulder straps by pulling down on the loose webbing.

\* Do not shorten the shoulder straps too much. This can cause the lap belt to be over the stomach causing injury.

9. Clip chest clip, and position at armpit level.