



**EZ-ON  
PRODUCTS**

# 403PB

## PUSH BUTTON ADJUSTABLE VEST WITH LOOPS

### INSTALLATION INSTRUCTIONS

#### IMPORTANT INFO

- Meets FMVSS 213 when used with a compatible EZ-ON mounts; model 100SMU, 100FM, 100TM.
  - Do not use with non-EZ-ON mounts
- For passengers over 31lbs
- Vests sizes XS-M come with crotch straps meeting NHTSA guidelines. Crotch straps on size Large and customs must be specifically requested.
- When heavy clothing, like winter jackets, are worn, the vest must be worn underneath.
- Available accessories; neck pads, wrists straps and upper arms straps
- Recommended replacement: 6 years after date of manufacturing.
- Sizing according to chest measurements
  - XS 18"-24" | S 25"-32" | M 32"-37" | L 37"-43"
- Custom Sizing available



For passengers over 31 lbs



#### WARNING

Death or serious injury can occur if manufacturer's instructions are not followed correctly. Read below warning information for replacement recommendations.



#### REGISTER

Register this transportation vest to be notified of a safety recall.



#### WARRANTY

EZ-ON Products stands behind the products we make. Warranty covers any defects in material or workmanship under normal use during the warranty period. We will repair or replace product defects for 1 year from manufacturer date.


WARNING: All EZ-ON products are dynamically tested to meet NHTSA standards. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement period is six years. EZ-ON Products are not liable for injury if these recommendations are not followed.

## PINCH TEST PROCEDURE

Tighten vest so that no pinch in the webbing can be achieved on the chest strap. When vest is snug fingers will be unable to grab any webbing.



**INCORRECT**

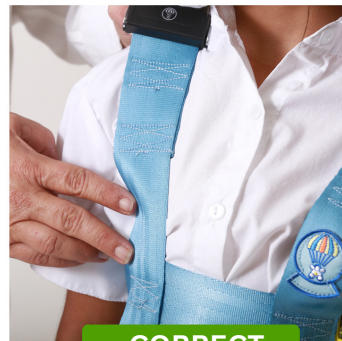

**CORRECT**

605 Commerce Way West  
PO Box 660  
Jupiter, FL 33458

**561.747.6920**  
cs@ezonpro.com

**EZONPRO.COM**



**EZ-ON  
PRODUCTS**

# 403PB

SCAN CODE TO VIEW VIDEO  
INSTRUCTIONS & MORE



## INSTALLATION INSTRUCTIONS



1. Hold vest while passenger steps into crotch straps with the E-Z-ON® parachute logo to front.



2. Keep bottom strap low around hip level.



3. Close and secure push buttons.



4. Pull webbing ends snug.



5. Adjust crotch straps to shorten or lengthen to be snug but not tight.



6. Adjust shoulder straps using adjusters. Pull webbing through adjuster to reduce slack.

*Note: Do not pull too tight as this will cause vest to pull up off of hips resulting in injury.*

### KEY STEP

Back thread webbing through bar slides to prevent vest from loosening. Proceed to follow manufacture instructions for the EZ-ON compatible mount.



*Exceptional Safety Solutions for Transporting Children and Adults.*