# MAXPV MOUNT



### **INSTALLATION INSTRUCTIONS**

#### **IMPORTANT INFORMATION**

- Meets FMVSS 213 when used with a compatible EZ-ON vest; model MaxPV.
  Do not use with non-EZ-ON vests.
- Installation of 100MPV requires the use of a tether anchor.
  - Use the vehicle's factory-installed (or EZ-ON's tether hardware kit) tether anchor.
  - If the vehicle manufacturer states a tether anchor weight limit and the child exceeds that weight, install/use EZ-ON's tether hardware kit (unless the vest is used in Positioning-Device Mode, per instructions, as described below).
  - Instructions for EZ-ON's hardware kit are included on the kit's assembly bag; video instructions are also available at ezonpro.com.
- Recommended replacement: 6 years after date of manufacturing.
- Passengers using EZ-ON transportation vest systems that include this mount:
  - Must weigh 31 lbs-168 lbs; if over 168 lbs, see Positioning-Device Mode, below.
  - Must utilize a seat belt, per vest instructions.
  - Should not ride in a booster seat.
  - May use Positioning-Device Mode if height is 4'9" or taller. In this mode, the occupant's vest is attached to the mount for support and the vehicle lap-shoulder belt is worn, following vehicle manufacturer instructions, over the vest. (Note: Passengers who weigh more than 168 lbs. and meet the height requirement (4'9") must use this mode; shorter passengers who weigh over 168 lbs. should not use this product.



WARNING: All EZ-ON products are dynamically tested to meet NHTSA standards. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement period is six years. EZ-ON Products are not liable for injury if these recommendations are not followed.

EMERGENCY RELEASE

CUT ALONG DOTTED LINE FOR EMERGENCY RELEASE





605 Commerce Way West PO Box 660 Jupiter, FL 33458 561.747.6920 cs@ezonpro.com

# EZONPRO.COM



# **MAXPV MOUNT**

MOUNT FOR PERSONAL VEHICLES

### **INSTALLATION INSTRUCTIONS**

## SCAN CODE FOR VIDEO INSTRUCTIONS & MORE



#### **EZ-ON PRODUCTS**



1. Fold seat and locate tether anchor. May have to remove fabric cover to gain access to bite of seat



6. Feed bottom tether hooks through bite of seat



2. Lay mount over back of seat



3. Connect central tether hook to lower vehicle anchor



8. Properly seat the passenger



4. Disconnect shoulder straps from D-ring and hang straps over the seat



9. Loop bottom tether hooks through hip D-rings on vest



5. Disconnect bottom tether hooks from D-ring



10. Feed bottom tether hook back through bite of seat. Captain's chairs, can go around side of the seat. Repeat step 9-10 for opposite side



11. Starting from the top down, loop shoulder tether hooks through shoulder D-rings and lay strap over the back of the seat. Repeat this step for the opposite side

7. Fold seat up and pull

bottom tether hooks fully

through bite of seat

12. Hook bottom tether hook to lower D-ring on mount



 Tighten bottom tether hooks by pulling on loose webbing through tilt adjuster. Ensure it is fully tightened before proceeding to next step



14. Hook shoulder tether hooks to the top D-ring on the mount



15. Tighten shoulder straps by pulling the loose webbing through the tilt adjuster

If passenger is over 4'9," may use seatbelt as intended by vehicle manufacturer. Skip steps 7-9



16. Thread vehicle seat belt through vest loops



17. Buckle seatbelt low on the lap. Make sure seatbelt is in the locked position and secured as tightly as possible



#### Exceptional Safety Solutions for Transporting Children and Adults.