

# MAXPV VEST

## **INSTALLATION INSTRUCTIONS**

### **IMPORTANT INFO**

- Meets FMVSS 213 when used with a compatible EZ-ON mounts; model 100MPV.
  - Do not use with non-EZ-ON mounts
- For passengers over 31 lbs
- Vests in all sizes come with crotch straps meeting NHTSA guidelines.
- When heavy clothing, like winter jackets, are worn, the vest must be worn underneath.
- Available accessories; neck pads, wrists straps and upper arms straps
- Recommended replacement: 6 years after date of manufacturing.
- Sizing according to chest measurements

correctly. Read below warning information for

replacement recommendations.

- XS 18"- 24" | S 25"- 32" | M 32"- 37" | L 37"- 43" | XL 43"- 49"
- Custom Sizing available



#### For passengers over 31 lbs



warranty period. We will repair or replace product

defects for 1 year from manufacturer date.



WARNING: All EZ-ON products are dynamically tested to meet NHTSA standards. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement period is six years. EZ-ON Products are not liable for injury if these recommendations are not followed.

# **PINCH TEST PROCEDURE**

Tighten vest so that no pinch in the webbing can be achieved on the chest strap. When vest is snug fingers will be unable to grab any webbing.



605 Commerce Way West PO Box 660 Jupiter, FL 33458 561.747.6920 cs@ezonpro.com

# **EZONPRO.COM**



# **MAXPV VEST**

#### SCAN CODE TO VIEW VIDEO INSTRUCTIONS & MORE

### **INSTALLATION INSTRUCTIONS**





1. Hold vest while passenger steps into crotch straps with the EZ-ON® parachute logo in front.



3. Hook zipper lock into zipper pull. Keep bottom strap low around the hip level.



5. Adjust shoulder straps using adjusters. Pull webbing through adjuster to reduce slack. Do not pull too tight as this will cause vest to pull up off hips resulting in injury.



2. Choose one of the (3) zippers that fits child snuggly. Be certain that the vest fits secure.



4. Adjust crotch straps to shorten or lengthen, to be snug but not tight. Keep the bottom strap low around the hip level.



6. Back thread web through bar slides to lock.

### Exceptional Safety Solutions for Transporting Children and Adults.