

86Y-CAR CAM

INSTALLATION INSTRUCTIONS

IMPORTANT INFO

- Meets FMVSS213
- Installation of 86Y requires the use of a tether anchor.
 - Use the vehicle's factory-installed (or EZ-ON's tether hardware kit) tether anchor.
 - If the vehicle manufacturer states a tether anchor weight limit and the child exceeds that weight, install/use EZ-ON's tether hardware.
 - Instructions for EZ-ON's hardware kit are included on the kit's assembly bag;
 video instructions are also available at ezonpro.com.
- 100H (Hardware Kit) not included.
 - If you know you will need the 100H, it must be requested at the time of purchase. However, it can be purchased on its own if you already have the product.
- Passengers using EZ-ON transportation vest systems 86Y:
 - Must weigh over 66 lbs.
 - Must utilize a seat belt, per vest instructions.
 - 。 Should not ride in a booster seat.
- When heavy clothing, like winter jackets, are worn, the vest must be worn underneath.
- Recommended replacement 6 years after date of manufacturing.



WARNING: All EZ-ON products are dynamically tested to meet NHTSA standards. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement period is six years. EZ-ON Products are not liable for injury if these recommendations are not followed.

CUT ALONG DOTTED LINE FOR EMERGENCY RELEASE



605 Commerce Way West PO Box 660 Jupiter, FL 33458

561.747.6920 cs@ezonpro.com

EZONPRO.COM



For passengers over 66 lbs



86Y CAR CAM

SCAN CODE TO VIEW VIDEO INSTRUCTIONS & MORE

INSTALLATION INSTRUCTIONS



Check the vehicle owner's manual to locate factory installed tether bracket. 43YC tether strap can be turned one half twist to snap on difficult anchor locations.



1. Locate the tether anchor bracket.



4. Adjust tilt lock adjuster to remove excess webbing so Y-bridge sits on seat top.



7. Thread the vehicle seat belt across the lap and through each belt loop.



2. Connect tether hook to tether anchor bracket directly behind passenger. Twenty degrees from "Y" center is acceptable.



5. Place the adjustable belt straps over each shoulder of the passenger.



8. First adjust the gray crotch strap from the top of the seat, by pulling on the loose webbing.



9. To adjust the shoulder vest straps, slide the webbing up or down through the adjuster. Keep both adjusters as close to the shoulders as possible.



3. Place the Y-bridge on the seat top. Rest black shoulder straps on the front of the seat.



6. Allow the belt loop ends to lay on the passenger's lap.



10. Secure the chest clip keeping the clip below the vest adjusters at arm pit level.

WARNING: Do not shorten the shoulder straps too much. This can cause the lap belt to be over the stomach causing injury.

Exceptional Safety Solutions for Transporting Children and Adults.