



# 100AR

## ANKLE STRAPS

### IMPORTANT INFO

- Must be used under direct supervision.
- Hinder unwanted leg movements.
- Use with EZ-ON Vest only.
- Register this product to be notified for a safety recall.



### WARRANTY

*EZ-ON Products stands behind the products we make. Warranty covers any defects in material or workmanships under normal use during the warranty period. We will repair or replace product defects for 1 year from manufacturer date.*

### WARNING

Death or serious injury can occur if manufacturer's instructions are not followed correctly. All EZ-ON products are dynamically tested to meet NHTSA standards. Please follow washing instructions on product tag. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement after six years. EZ-ON Products are not liable for injury if these recommendations are not followed.

### INSTALLATION INSTRUCTIONS



1. Wrap strap around ankle one at a time.
2. Clip buckle closed one at a time.
3. Adjust the size by pulling on excess webbing.
4. Feed excess webbing through plastic barslide.
5. Clip the buckle connecting the ankles together.