



EZ-ON PRODUCTS

1-800-323-6598

www.ezonpro.com

225A

LOWER LATCH ATTACHMENTS

IMPORTANT INFO

- Use only with EZ-ON Vests. Compatible vests include: 103Z, 403PB, 303Z, and 203PB.
- Must be used with EZ-ON Tether Mount (100TM)
- No seat belt is necessary.
- Check your vehicle manual to identify latch seating positions, anchor locations, and weight limits.
- Register this product to be notified for a safety recall.




WARRANTY

EZ-ON Products stands behind the products we make. Warranty covers any defects in material or workmanships under normal use during the warranty period. We will repair or replace product defects for 1 year from manufacturer date.

WARNING

Death or serious injury can occur if manufacturer's instructions are not followed correctly. All EZ-ON products are dynamically tested to meet NHTSA standards. Please follow washing instructions on product tag. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement after six years. EZ-ON Products are not liable for injury if these recommendations are not followed.

INSTALLATION INSTRUCTIONS

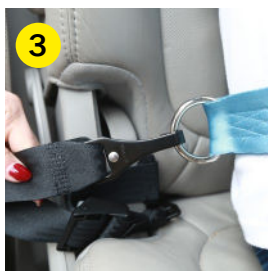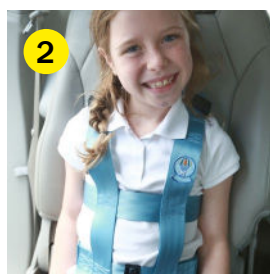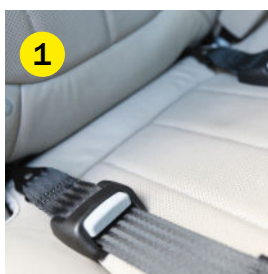

1. Attach the snap hooks on the 225A to the vehicles lower latch anchors. Put the EZ-ON Vest on the passenger per the manufacturer instructions.
2. Seat the passenger. The passenger's bottom should be as close as possible to the seat back.
3. Attach the snap hooks to the hip level D-Rings on the EZ-ON Vest.
4. Adjust the 225A by pulling on the loose webbing. Webbing should be tight.

* Passenger should not be able to move their bottom more than 2 inches in any direction.

* Proceed to follow manufacture instructions on the EZ-ON Tether Mount (100TM).



EMERGENCY RELEASE

In the event of an emergency 2 cuts need to be made. One cut of the chest strap, and one cut of the hip strap of the vest. Vest will open and the passenger can be removed.